

SARS-CoV-2

SARS-CoV-2 rapid test kit MACCURA



Severe Acute Respiratory Syndrome Coronavirus 2(SARS-CoV-2) IgM/IgG Antibody Assay Kit by Colloidal Gold Method

IgM and IgG 2-in-1

Guarantee of material optimization

Quick interpretation of results

Simple and convenient operation

Product Feature

IgM and IgG 2-in-1

2 results from 1 test, which can simultaneously determine negative/positive status of both IgM and IgG.

The guarantee of material optimization

The optimized protein fragments N and S of SARS-CoV-2 can improve detection capacity.

Quick interpretation of results

Under normal circumstances, the results can be observed within 10~15 minutes.


After 15 minutes, the results are invalid.

Strong positive results can be observed in 150 seconds.

Simple and convenient operation

Evaluating results without the aid of instruments

Operation Procedure

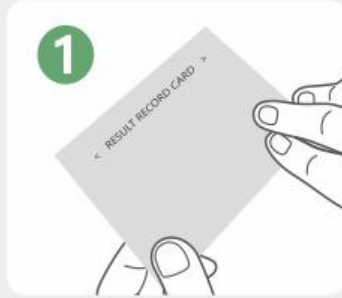

1
Take out the Result Record Card and place it horizontally on the table



2
Tear off the tape area with the **OPEN** mark



3
Take out the test strip carefully, and then immediately cover the lid to avoid the other test strips getting wet



4
Paste the test strip on the dotted area (Forbid to press on the loading area)
And write the ID number on it



5
Absorb the specimen



6
Add specimen vertically at the top of the loading area




7
Add Diluent vertically at the bottom of the loading area



8 **Wait 10-15min**
Wait 10-15 minutes to observe the result

Result Interpretation

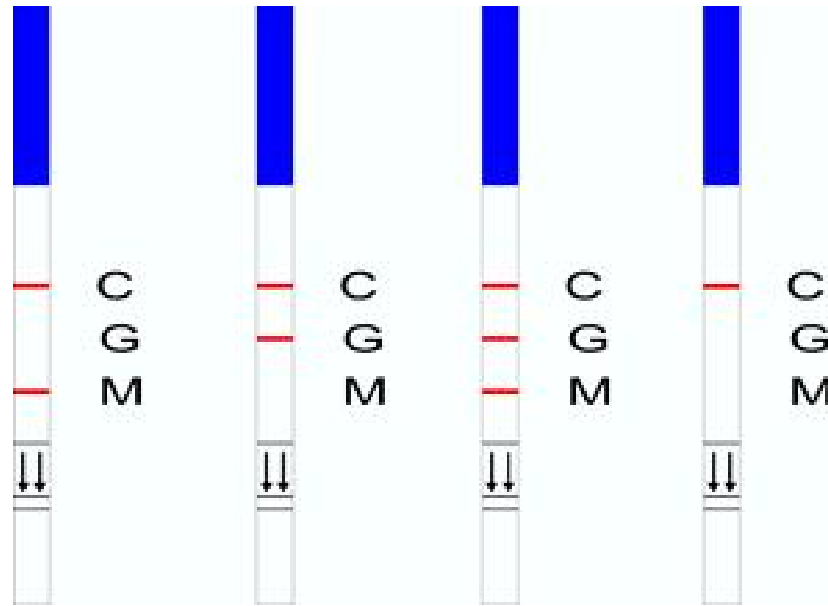


IgM
Positive

IgG
Positive

IgM, IgG
Positive

Negative

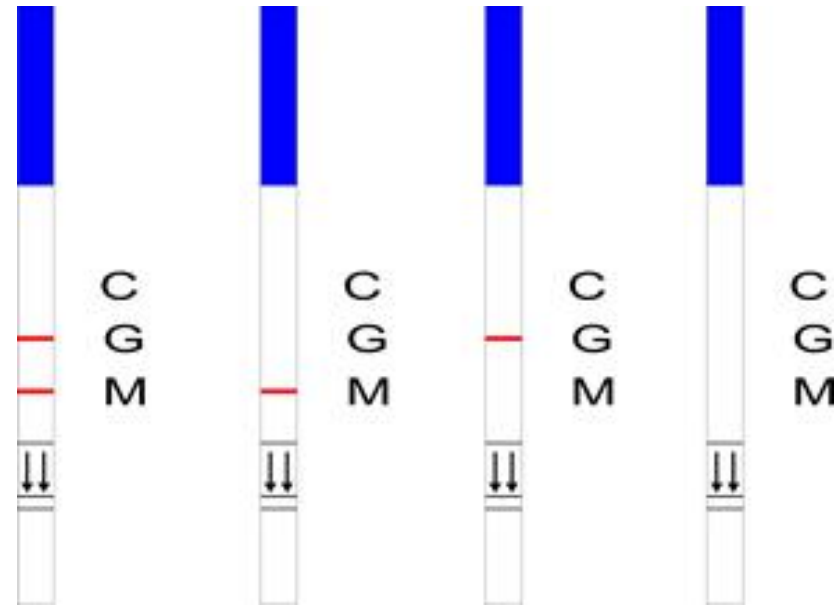

Invalid

Invalid

Invalid

Invalid

Production Information

Product name	Specification	Catalog No.
Severe Acute Respiratory Syndrome Coronavirus 2(SARS-CoV-2) IgM/IgG Antibody Assay Kit by Colloidal Gold Method	50 Tests/Tube ×2	EIM4103080